**Instructions for use**  
English

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# Charger

**SIEMENS**

English	3
Deutsch	17
Français	29
Italiano	41
Nederlands	53

## Contents

<u>Your charger</u>	4
<u>Intended use</u>	5
<u>Important notes</u>	6
<u>Notes for batteries</u>	8
<u>How to charge</u>	9
<u>Useful tips for charging</u>	12
<u>Maintenance and care</u>	13
<u>Disposal information</u>	14
<u>Technical data</u>	15
<u>Technical data – Country-specific power-supply</u>	16

EN


DE

FR


IT

NL


## Your charger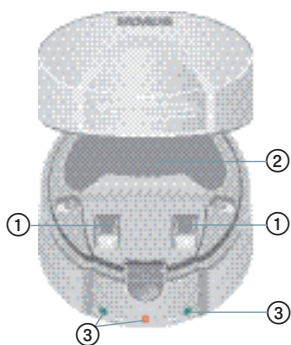

- ① Charging slots with charging contacts
- ② Cavity for receiver units, LifeTubes and domes
- ③ LEDs indicate the functional states of the charger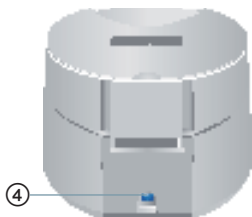

- ④ Contact for power supply

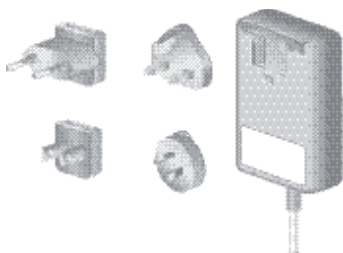- ⑤ Power supply with country-specific adapter

## Intended use

This charger is intended to charge Siemens hearing instruments.

Read and follow the instructions of this user guide to avoid injuries or damage to the charger and the hearing instrument.

EN

DE

FR

IT

NL

### Symbols:

Be aware of information marked with the warning symbol **WARNING**, **CAUTION** or **NOTICE**!

**WARNING** points out a situation that could lead to serious injuries,  
**CAUTION** indicates a situation that could lead to minor and moderate injuries.

NOTICE

**NOTICE** indicates possible property damage.

Advice and tips on how to handle your charger better.

Instruction. Indicates that something has to be done.

## Important notes

### WARNING

Hazard of explosion!

- ▶ Do not use the charger near flammable mixtures.

### WARNING

Choking hazard posed by small parts.

- ▶ Keep batteries and hearing instruments out of children's reach.
- ▶ If swallowed consult a physician or a hospital immediately.

### CAUTION

Risk of malfunction through incompatible parts.

- ▶ Use only the power-supply and country-specific adapter that is provided with the charger.
- ▶ Charge only Siemens rechargeable hearing instruments.

## Important notes

### CAUTION

Risk of malfunction through wrong voltage.

- ▶ Make sure that voltage corresponds to the technical data declared on the adapter.

- This product does not have a power switch. It is turned on when the power cord is plugged in.
- The charger is intended only for indoor use.

EN

DE


FR

IT

NL

## Notes for batteries

- ▶ Use only rechargeable batteries that are provided by your Hearing Care Professional.
- ▶ For information about battery size check the user guide of your hearing instruments.
- ▶ Make sure the rechargeable batteries are inserted correctly.
- ▶ Charge the rechargeable battery before first use.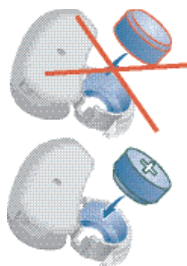

### NOTICE

Batteries contain harmful substances that pollute the environment.

- ▶ Do not throw used batteries into household trash.
- ▶ Return them to your Hearing Care Professional for environmentally correct disposal.


## How to charge

### Preparation

- ▶ Connect the charger to a power supply.

### Charging

- ▶ Press the front button to open the charger.  
The lid unlocks.
- ▶ Place your hearing instruments into the charging slots.
- ▶ Close the lid of the charger.  
The hearing instruments are switched off automatically and the charging process starts.  
The green LED is blinking.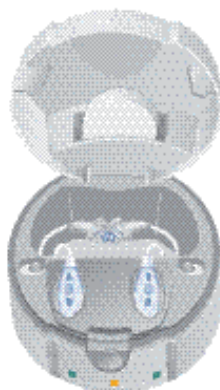

EN

DE

FR

IT

NL

Charging is complete when the green LED stays on continuously.

- ▶ Remove the hearing instruments from the charger when you next want to use them.
- ▶ Check if the hearing instruments are turned on.

## How to charge

The charger's LEDs indicate the charging status.

**Middle LED:** Connection to the power supply

**Left/right LED:** Charging process

LED and state	Explanation
Middle LED shining orange	Charger is connected to power supply. Charger is ready to use.
Middle LED is off	▶ Check whether charger is connected to power supply.
Left/right LED flickering green	Beginning of charging process. Charger checks hearing instruments and battery polarity.
Left/right LED blinking green	Charging process ongoing. ▶ Leave hearing instruments in charger.
Left/right LED shining green	Charging process finished. Charger preserves the charge of the battery. ▶ Remove hearing instruments from charger when you want to use them.
Left/right LED shining red	Charging process cannot be initiated. ▶ Check type and polarity of batteries. ▶ Check whether hearing instruments are inserted correctly in charging slots. ▶ Remove and reinsert power plug.

## How to charge

If type and polarity of batteries are okay:

- ▶ Ask your Hearing Care Professional to clean the contacts of the hearing instruments and charger.
- ▶ Ask your Hearing Care Professional to check the functioning of the rechargeable batteries and the charger.

Consult your Hearing Care Professional if you encounter further problems.

EN

DE

FR

IT

NL

## Useful tips for charging

- ▶ Wear your hearing instruments a full day.
- ▶ Charge the rechargeable batteries over night.  
A charging cycle takes about 5 hours.

With this usage one rechargeable battery has a life time of about 1,5 years.

When a charging cycle is disrupted (e.g. by opening the lid of the charger) a new charging cycle starts.

**i** If you need to remove the hearing instruments before the charging cycle is complete, the operation time can be shorter than usual.

To maximize the lifetime of your rechargeable batteries:

- ▶ Avoid to disrupt an ongoing charging cycle.
- ▶ When the hearing instruments are fully charged and you do not want to use them, leave them in the charger.

## Maintenance and care

EN

DE

FR

IT

NL


### NOTICE

The charger will be damaged by dissolver.

- ▶ Use a dry, soft tissue to clean the charger.
- ▶ Do not sterilise or disinfect the charger.

### Dry function

- ▶ Place a drying capsules into the holder located inside the lid of the charger.
- ▶ Leave the lid of the charger closed when possible to extend the lifetime of the drying capsule.
- ▶ Exchange the drying capsule when its yellow content has turned white.

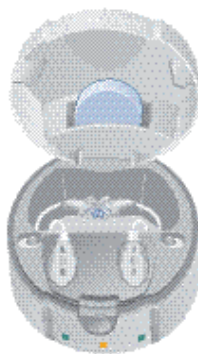### CAUTION

Hazard of poisoning, when swallowing content of drying capsule.

- ▶ Keep drying capsule out of children's reach. If swallowed consult a physician or hospital immediately.

## Disposal information

Within the European Union, the marked equipment is covered by "Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electronic equipment.

Amended by Directive 2003/108/EC" (WEEE).

 ► Recycle the charger according to national regulations.

[www.siemens.com/hearing](http://www.siemens.com/hearing)

With the CE marking Siemens confirms compliance with the European Directive 2004/108/EC concerning electromagnetic compatibility.

## Technical data

### Conditions for storage, transportation and operation

- i** ▶ Do not expose the charger to extreme temperatures or high humidity.
- ▶ Do not expose to direct sunlight.

<b>Operating temperature:</b>	15 °C to 35 °C
<b>Operating humidity:</b>	20% to 75%
<b>Minimum air pressure:</b>	70 kPa
<b>Storage temperature:</b>	-10 °C to 40 °C
<b>Storage humidity:</b>	20% to 75%

### Specifications charger

<b>Input voltage:</b>	100-240 V AC
<b>Input current:</b>	<1.2 A
<b>Protection against electric shock:</b>	Class II
<b>Safety approvals:</b>	cULus, CE, C-Tick
<b>EMI:</b>	Complies with EN 60601-1-2, EN 55011, Class B
<b>Against ingress of water:</b>	IXPO (ordinary)

EN

DE

FR

IT

NL

## Technical data – Country-specific power-supply

### Model GT41076-0605, Globtek

<b>Input voltage:</b>	100-240 V AC
<b>Input current:</b>	>0.6 A RMS max
<b>Input frequency:</b>	47-63 Hz
<b>Output voltage:</b>	5 VDC
<b>Output current:</b>	1.2 A
<b>Output power (rated):</b>	6 Watt max
<b>Against electric shock:</b>	Class II

### Model GTM41076-0605, Globtek

<b>Input voltage:</b>	100-240 V AC
<b>Input current:</b>	>0.6 A RMS max
<b>Input frequency:</b>	47-63 Hz
<b>Output voltage:</b>	5 VDC
<b>Output current:</b>	1.2 A
<b>Output power (rated):</b>	6 Watt max
<b>Against electric shock:</b>	Class II

### Model Fuhua Electronic Co Ltd

<b>Input voltage:</b>	100-240 V AC
<b>Input current:</b>	0.4 A Max.
<b>Input frequency:</b>	50-60 Hz
<b>Output voltage:</b>	5 VDC
<b>Output current:</b>	max. 0.5A
<b>Output power (rated):</b>	2.5 Watt max
<b>Against electric shock:</b>	Class II

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© Siemens AG, 05.2009 · ANR
Siemens Audiologische Technik GmbH